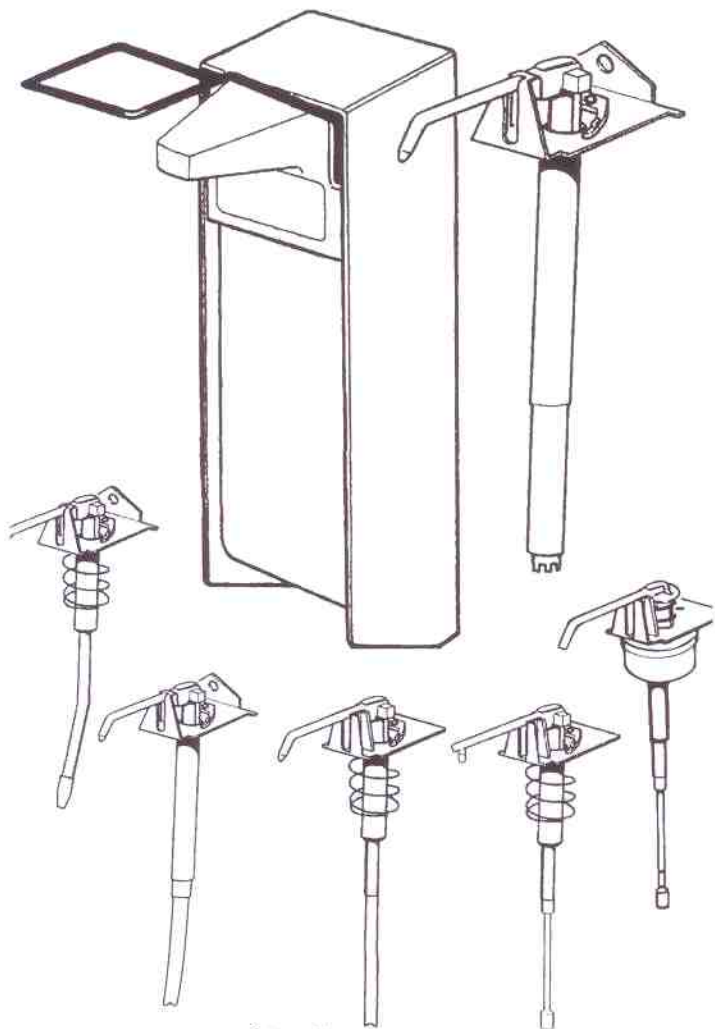
# ingo-man® 26

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Dispenser for soap and disinfectant  
with interchangeable pump units

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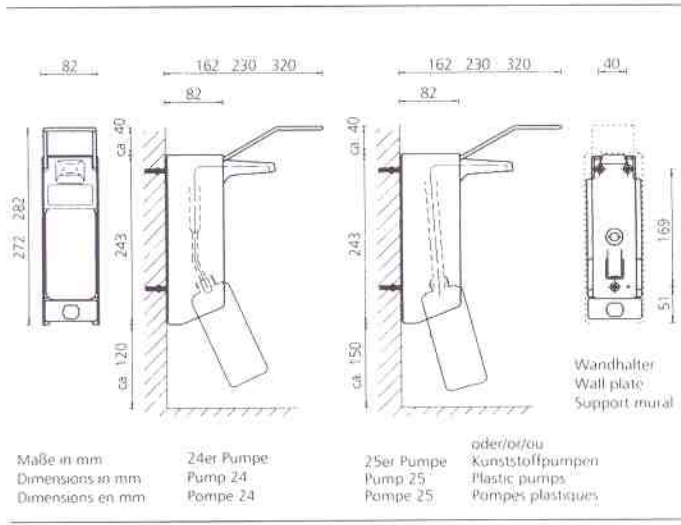
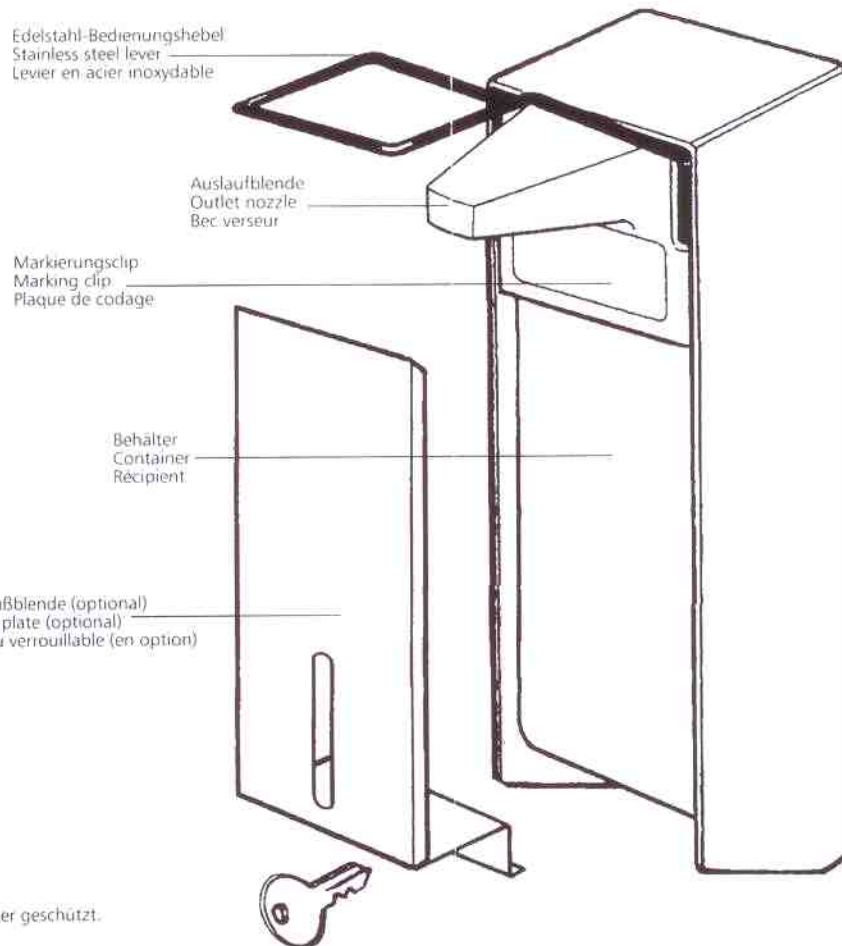
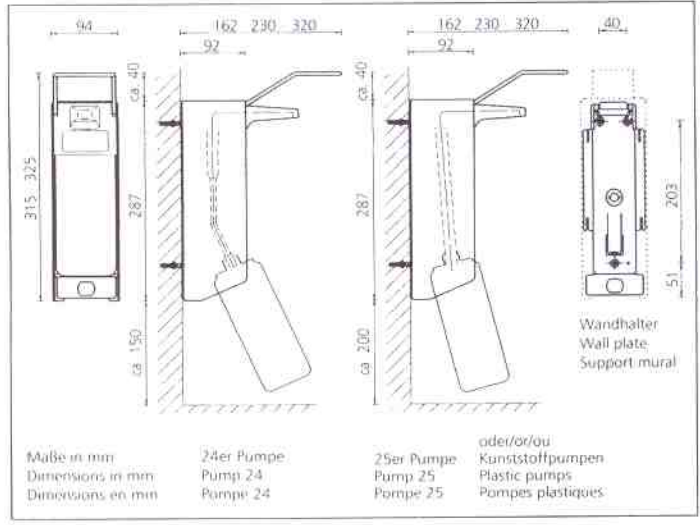
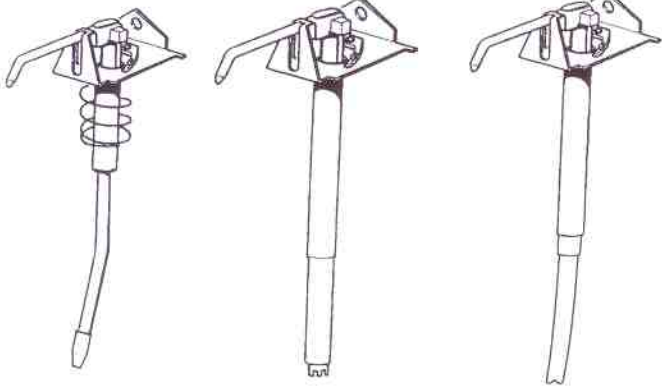
Range 24 pump with crooked stainless steel sucking tube, autoclavable

Range 25 pump with straight stainless steel sucking tube, autoclavable

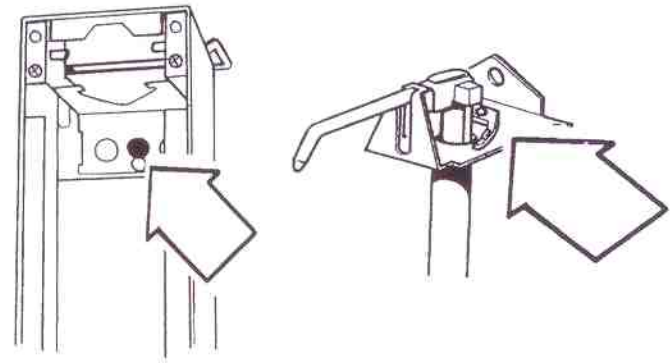
Range 28 stainless steel pump with flexible sucking tube, autoclavable

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
Each unit is colour coded inside of the chassis. The pumps are marked by a coloured dosage fork (except foam pump, see below). The colour indicated on the base unit must be same as of the dosage fork.

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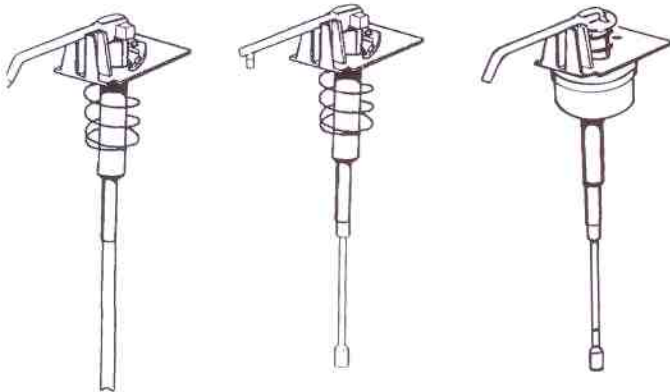Plastic pump with straight plastic sucking tube, non autoclavable

Spray pump for disinfectant, plastic non autoclavable

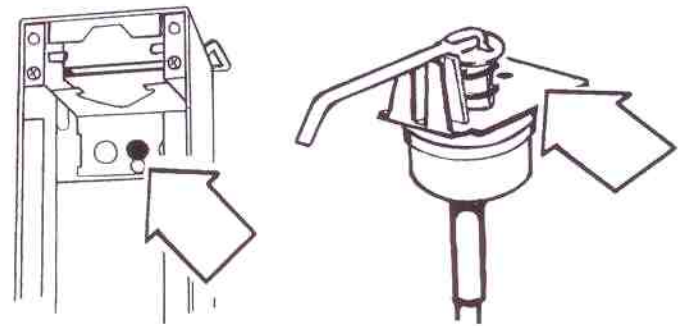
Pump for foamy soap, plastic, non autoclavable

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


Each unit is colour coded inside of the chassis, also the foam pump. The colour indicated on the base unit must be same as on the foam pump.

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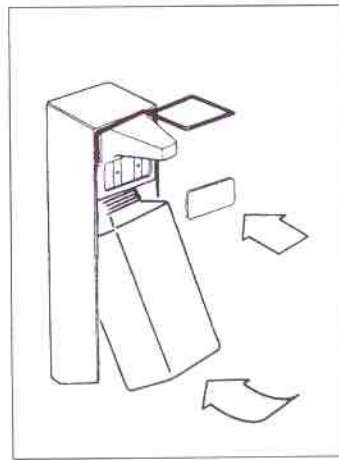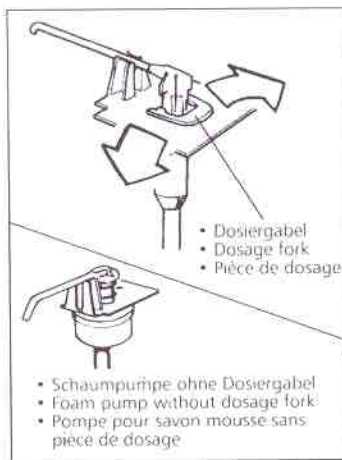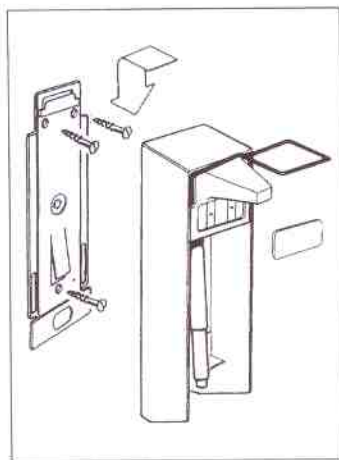#### Check before installation

1. The dispenser should be fitted above a sink so that wet hands can drip.
2. Required space above the dispenser is 40 mm; for below refer to dimensions overleaf.
3. Required height: 130–135 cm, from floor to top of dispenser.

#### Mounting

Remove the container. Slightly press the fixing catch and pull down the wall plate. Fix wall plate with enclosed plugs and screws. Mount the dispenser onto the wall plate by firmly pressing until it clicks into place.

#### Dosage

The dispenser is adjusted to the maximum amount of approx. 1.8 ml/stroke. If required, the dosage can be reduced to a minimum of approx. 0.8 ml/stroke by gradually turning the dosage fork head. **The foam pump has not a dosage fork. The dosage is not changeable.**

#### Container exchange

Insert the refilled container from below and press the facial clip into position.

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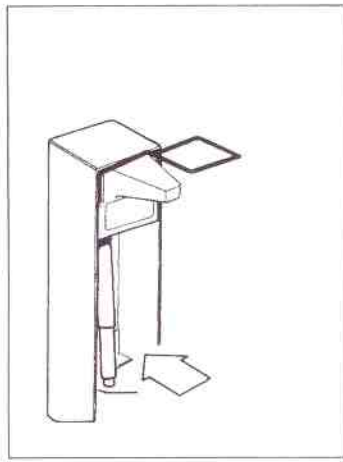
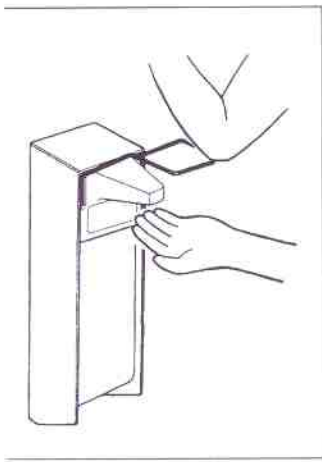
#### Initial operation

At the first start-up, the locking lever must be pushed up several times until the dispenser dispenses. Hold the open hand under the outlet nozzle.

#### Operation

At initial working, the unit must be operated several times until it delivers fully. Hold the open hand under the outlet nozzle.

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#### Cleaning/Sterilizing

Press locking device and push dispenser upwards. Dispensers using aluminium- and stainless steel for units fitted with a stainless steel pump or stainless steel pumps alone (also the range 28 pump with flexible sucking tube) are autoclavable up to 121 °C, 1 bar.

**Plastic pumps and powder coated dispensers are not autoclavable!**

#### Cleaning/Sterilizing

Press locking device and push dispenser upwards. Dispensers using aluminium or stainless steel for units fitted with a stainless steel pump or stainless steel pumps alone (also the range 28 pump with flexible sucking tube) are autoclavable up to 121 °C, 1 Bar. **Plastic pumps and powder coated dispensers are not autoclavable!**

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#### Maintenance

The Robert-Koch Institute recommends: **“Dispensers for soap and disinfectants must be thoroughly cleaned or sterilised whenever the cartridges are changed”** [Source: see above “Quelle”]. – Though the pump is made from stainless steel, rust can occur. This can be prevented by observing the above recommendation. – Routine cleaning with warm water and soap, do not use any abrasive cleansers or such containing chlorine.

**Do not dismantle the unit. Use original spare parts only. Attention! If you fail to use the right liquid, it can splash into your eyes.**

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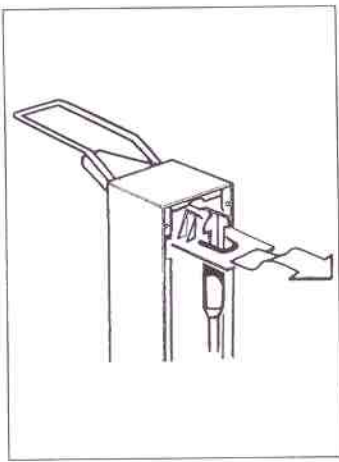
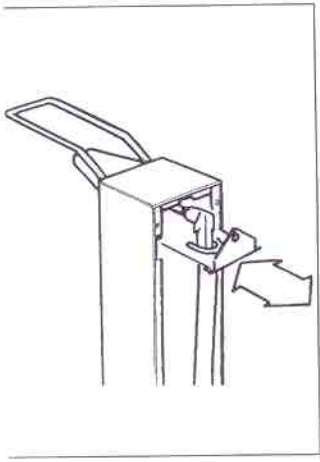
#### Pump exchange

Make sure that the colour code inside of the chassis is same as colour of the dosage fork.

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**Exchange of  
 stainless steel pump**

Hold operating lever up and pull  
 out stainless steel pump from  
 back. For inserting the stain-  
 less steel pump hold operating lever  
 up and place pump into position.

**Exchange of  
 plastic pump**

Hold operating lever up and slightly  
 lift up the plastic pump and pull it  
 out. For inserting the plastic pump  
 hold operating lever up and place  
 pump into position.

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